

ACCURO

100% IP telecommunications and security
platform for hospitals and nursing homes



ACCURO Demo Kit Start Guide

The DKA-231 suitcase is a portable model of the ACCURO nurse call system. It contains the typical elements that can be found in a hospital or nursing home room, and shows its operation mode:


- CHC-231 bedroom audio room terminal.
- Wall base with call buttons and handset connector: LLC-903.
- Call and lighting control handset: LLC-711.
- Toilet pull cord: LLC-921.
- Card reader for presence and access control: RFI-722.
- SEN-724: overdoor light.

In addition, it is equipped with a WIFI network which allows the calls reception on SIP telephones and to access ACCUROViewer and ACCUROManager softwares.

Here's how to start the system, access ACCUROViewer and ACCUROManager, and configure a smartphone to receive calls.

1. Plug & Switch on the suitcase

Wait for the CHC-231 to display only the following text (it may take up to 1 min):

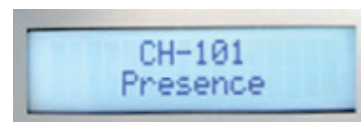

2. Access ACCUROViewer

Access the Wifi of the suitcase:
ACCURO_WIFI

Read the following QR code:

<http://172.23.201.1/ALCADViewer>



3. Access ACCUROManager

Access the Wifi of the suitcase:
ACCURO_WIFI

Read the following QR code:

<http://172.23.201.1/ALCADManager>

User: admin

Password: alcad



4. Configure the smartphone

In order to receive calls and be able to use the intercom with the room terminal, it is necessary to install a smartphone application:

We recommend the free APP, GS Wave.

Download the softphone APP.

Enter the APP

Go to: Settings -> Account Options -> Click on the "+" symbol.



Configure the account with the following data:

Activate Account	<input checked="" type="checkbox"/>
Account Name	1001
SIP Server	172.23.201.1
SIP User ID	1001
SIP Authentication ID	1001
Password	**** 1234 ****
Voicemail UserID	
Display Name	

Save.